

# How to use the Deep Margin Elevation Wedges

We have included some complimentary Deep Margin Elevation Wedges (DME Wedge) for you to try in your QuickConnect Anatomy Band Kit. Deep Margin Elevation Wedges (formally known as the OVC Wedge) provide an amazing anatomical shape and allow predictable deep margin elevation. Also, they are a great option for cementing crowns and large fillings.



Turn over for quick instructions on how to use the DME wedge.

Visit [www.rhondium.com/dme-wedge/#video](http://www.rhondium.com/dme-wedge/#video) or scan this QR code to view instructional video



1. Place DME Wedge with the smooth face toward the deep cavity. They can be placed either buccally or lingually. Sometimes switching between buccal or lingual placement can give you a better seal or shape.



2. Etch and bond, then use your preferred composite to build up the margin using the DME Wedge like a matrix band.



3. Light cure the composite and then remove the wedge. Now your margin is built up and ready for a matrix band.