




# QuickConnect Anatomy Band Tips

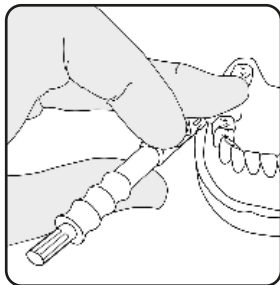

Best placement technique:  
Secure the matrix band using your middle finger. Firmly hold the carrier with your thumb and forefinger. Using your free hand, turn the Carrier Driver clockwise to tighten, or have your assistant do so.

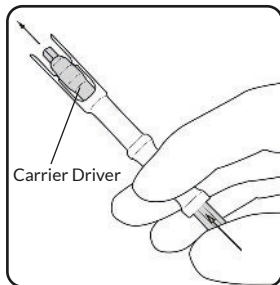

Push Carrier Driver forward like a syringe to attach and detach carrier from the QuickConnect Retainer.

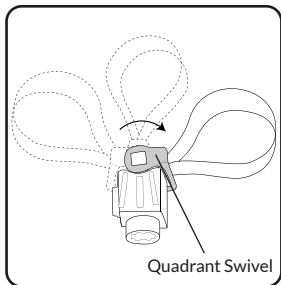


The Quadrant Swivel is used to swivel the band for use on any quadrant.

## **Need to push the band through a closed contact?**

Place a wedge between the closed contact to create separation before placing the matrix band.

# QuickConnect Anatomy Band QuickGuide

1. Push the carrier driver forward in the carrier sleeve to expand the carrier arms. Slide the carrier arms into the V-shaped grooves on the top and bottom of the plastic body of the retainer. This will cause the arms to lock onto the plastic body of the retainer.
2. If necessary, rotate the swivel to the appropriate side for the working quadrant.
3. Place the band over the tooth and tighten by rotating the carrier driver clockwise.
4. Push the carrier driver forward to **detach the carrier before restoring the tooth.**

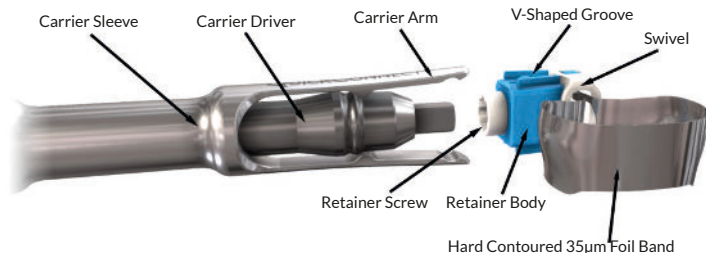


Visit [www.rhondium.com/QC/videos](http://www.rhondium.com/QC/videos) or scan this QR code for training videos



## What are the tabs for?

The “Tight-Contact-Tabs” are pulled with a dental probe while light-curing to ensure the matrix band is in contact with adjacent teeth, guaranteeing tight contacts. Alternatively, they can be temporarily bonded to the occlusal surfaces of the adjacent teeth using dental bond and flowable composite. This second option can save valuable time by allowing you to fill and cure both the mesial and distal boxes simultaneously. The flowable is removed with a strong hand instrument or No. 12 scalpel after the restoration is complete.

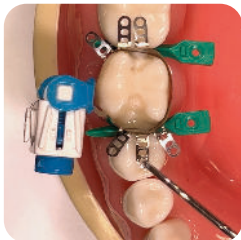

Option 1: Pull on tab while curing the proximal composite.



Option 2: Use flowable to tack tabs to adjacent tooth.

## Company details

EC

REP

MedEnvoy Global BV Prinses  
Margrietplantsoen 33 Suite 123  
2595 AM The Hague  
The Netherlands

### UKRP

MedEnvoy UK Limited  
85, Great Portland Street.  
First Floor London, W1W 7LT  
United Kingdom



Rhondium Ltd  
4B Sheffield St  
Katikati 3129  
New Zealand

Phone NZ: 0800 746 634  
Phone US: 1 800 875 6246  
Phone Australia: 1 800 449 282  
Phone UK: 0800 680 0636  
Phone Ireland: 1 800 817 177  
Phone International: +64 7 549 5566

Email: [info@rhondium.com](mailto:info@rhondium.com)  
Email UK/Ireland: [support@rhondium.com](mailto:support@rhondium.com)

**Cleaning Instructions:** QuickConnect Anatomy Bands: single-use, autoclavable for sterility. QuickConnect Carrier: autoclavable and disassembles for easy cleaning.